


How to export DICOM Images from i-CATVision and upload them to IDT Scans

Step 1

- Open **i-CATVision**
- Load the images to be sent to IDT Scans
- Click **Tools**
- Click **Export DICOM**

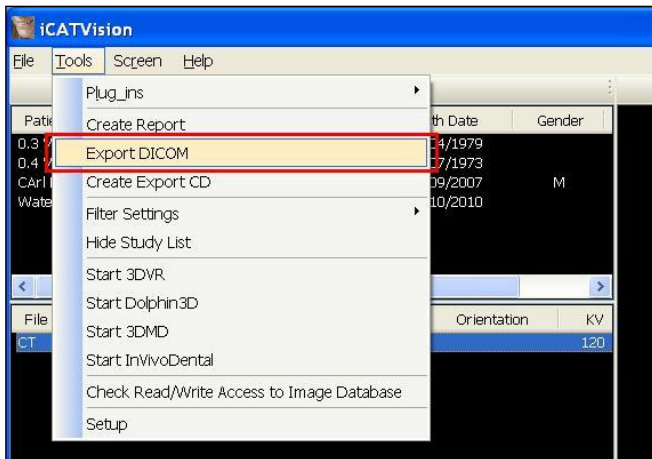

Step 2

In **DICOM Export Properties** ensure that **Single File DICOM** is selected. We recommend selecting **Compressed** as well.



Step 3

- Click **Browse** and select the **C:\DICOM Exports** folder (unless it is already selected)
- Click **OK** to start to export the DICOM data to a single file in the **C:\DICOM Exports** folder.
- The file will be called <Patient Name>.dcm or <Patient ID>.dcm or a similar name ending in “.dcm”.


Step 4


Use Method A or Method B or Method C or any other method to send the file to IDT Scans.

Method A: if you have an sFTP Account

You can drag the .dcm file into the “To_IDT” folder.

Step 5A

- Right-click on the Windows  button
- Click **Explore**
- Open **C:\DICOM Exports** folder.
- Find the file you made in **Step 3**
- On your Desktop, open **WinSCP**
- Drag the .dcm file you created into the **To_IDT** folder.
- When the .dcm file has finished copying over, delete it from **C:\DICOM Exports** folder (and delete any other files that may be in the C:\DICOM Exports folder).

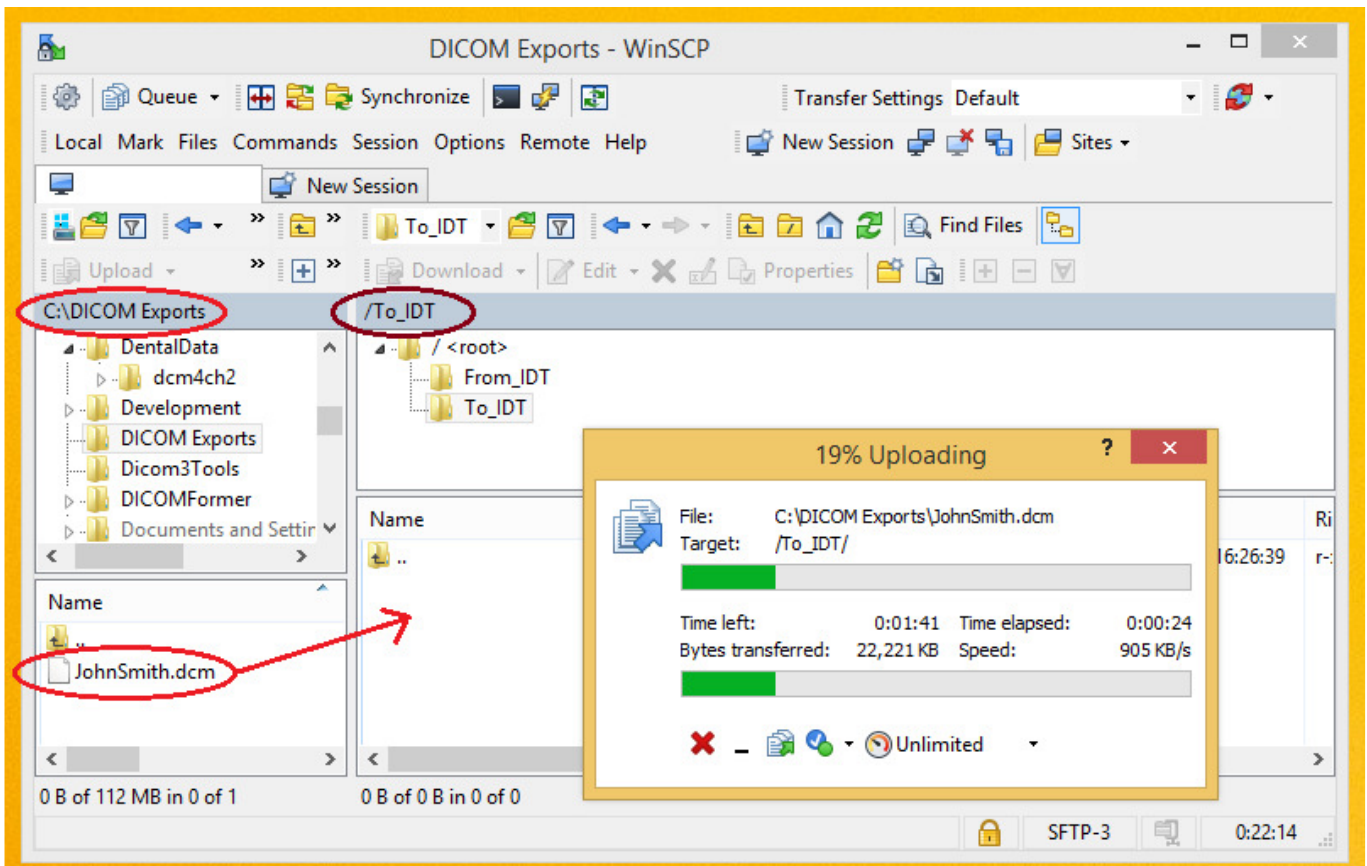


Method B: if you have registered on www.idtscans.com

You can upload the .dcm file while booking in the work you require.

Step 5B

- Go to www.idtscans.com
- Log into your account and start booking the case in
- select “.zip file containing DICOM images”

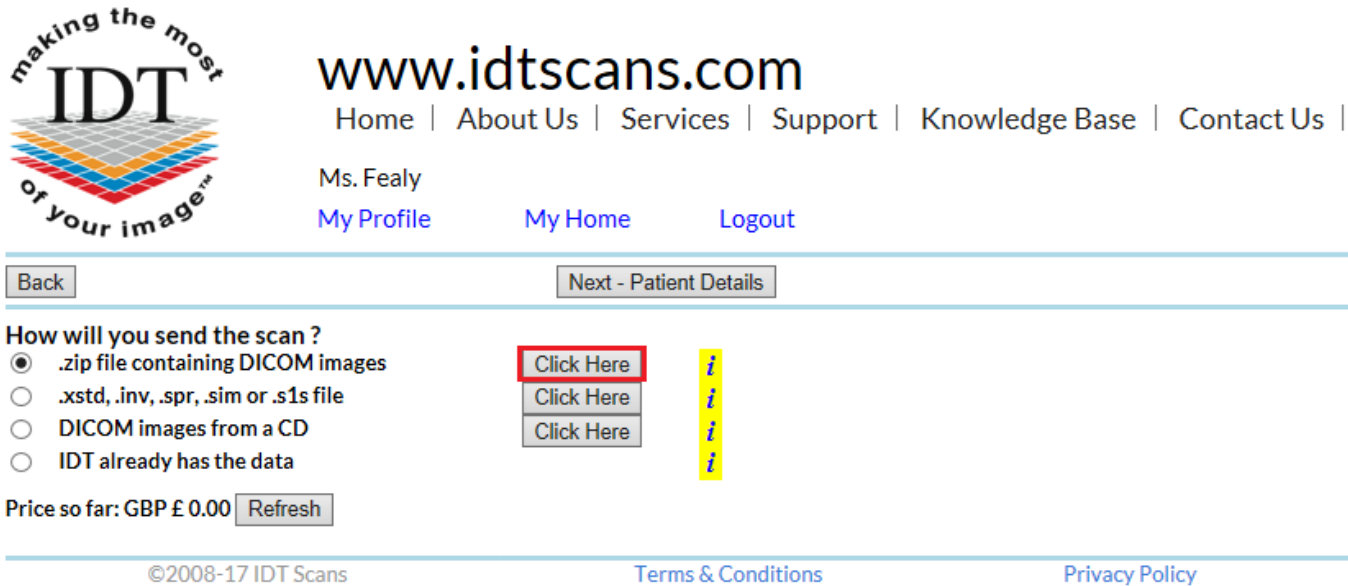


The screenshot shows the IDT Scans website. The logo on the left says "making the most of your image™" with "IDT" in the center. The main heading is "www.idtscans.com". Navigation links include "Home | About Us | Services | Support | Knowledge Base | Contact Us". The user is logged in as "Ms. Fealy" with links for "My Profile", "My Home", and "Logout". Below the navigation is a "Back" button and a "Next - Patient Details" button. The main section is titled "How will you send the scan ?" and has four radio button options:
• .zip file containing DICOM images (with a "Click Here" button highlighted in red)
• .xstd, .inv, .spr, .sim or .s1s file (with a "Click Here" button)
• DICOM images from a CD (with a "Click Here" button)
• IDT already has the data (with a "Click Here" button)
To the right of these options is a vertical yellow bar with three "i" icons. Below the options is a price display: "Price so far: GBP £ 0.00" with a "Refresh" button. At the bottom, there is a footer with "©2008-17 IDT Scans", "Terms & Conditions", and "Privacy Policy".

- attach and send your file (see below)
- carry on booking in the case while the file is being sent to IDT Scans.

Step 6B

Follow the 3 steps below to attach and send your file. If you cannot see the BROWSE button please install Adobe Flash Player from <http://get.adobe.com/flashplayer>

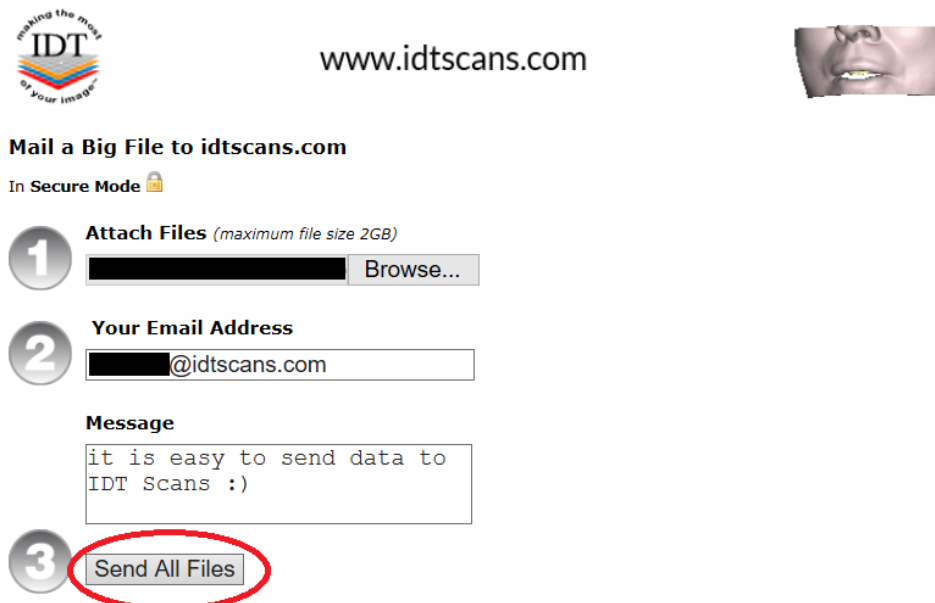

The screenshot shows the "Mail a Big File to idtscans.com" interface. It features the IDT Scans logo on the left and a partial image of a person's face on the right. The main heading is "Mail a Big File to idtscans.com" with a "In Secure Mode" indicator. The interface is divided into three numbered steps:
1. "Attach Files (maximum file size 2GB)" with a "Browse..." button.
2. "Your Email Address" with a text input field containing "████████@idtscans.com".
3. "Message" with a text area containing "it is easy to send data to IDT Scans :)".
Below the message is a "Send All Files" button, which is circled in red. At the bottom, there is a footer with "Developed & Hosted by Dessol | ©2005 - 2017 | Powered By MailBigFile.com | Switch to Multi Uploader" and flags for the UK, France, and Germany.

Method C: if you haven't registered yet

You can send the .dcm file first, then book in the work later.

Step 5C

Got to www.idtscans.com

If you haven't registered, click **Upload Your File Now**:



www.idtscans.com

[Home](#) | [About Us](#) | [Services](#) | [Support](#) | [Knowledge Base](#) | [Contact Us](#)

Get the most out of your dental CT/CBCT scans


REFORMAT AN EXISTING SCAN

REQUEST A RADIOLOGY REPORT

REQUEST A NEW DENTAL CT SCAN

IMPLANT SIMULATION, MODELS, GUIDES

UPLOAD YOUR FILE NOW

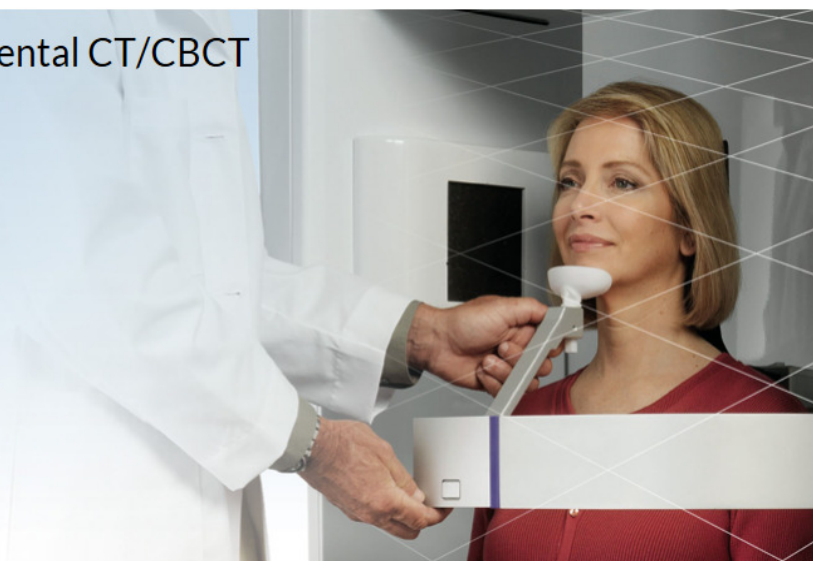

Step 6C

On the next page, click on “**Upload a Single File**”:



www.idtscans.com

[Home](#) | [About Us](#) | [Services](#) | [Support](#) | [Knowledge Base](#) | [Contact Us](#) |

Upload Files

If you are requesting work for IDT Scans to do, please click on one of the following links and follow the steps in the Online Booking wizard:

- [Reformat an Existing Scan](#)
- [Request a Radiology Report](#)
- [Implant Simulation, Treatment Planning, Guides](#)

If you want to send us a single file, please click here:

- [Upload a Single File](#)

If you want to send us multiple files, please place them in a folder. Right-click on the folder and select “Send to” and then select “Compressed (zipped) folder”. Then click on the following link and select the zipped folder:

- [Upload a Zipped Folder](#)

Please visit our extensive [Knowledge Base](#) for further information, or [Contact Us](#).

Step 7C

Follow the 3 steps below to attach and send your file. If you cannot see the BROWSE button please install Adobe Flash Player from <http://get.adobe.com/flashplayer>



www.idtscans.com



Mail a Big File to idtscans.com

In Secure Mode

1 **Attach Files** *(maximum file size 2GB)*

2 **Your Email Address**

Message

it is easy to send data to
IDT Scans :)

3

Developed & Hosted by Desso | ©2005 - 2017 | Powered By MailBigFile.com | [Switch to Multi Uploader](#)
MailBigFile ® registered trademark of MailBigFile Ltd