

How to Export an XSTD File from i-CATVision




XSTD files are the native format used by Xoran software on the i-CAT Classic.

Step 1

Launch i-CAT Vision and load the patient data.


Step 2

Go to **Tools->Plug_Ins** and click **XSTD Exporter** (if you can't see it, the XSTD Exporter plug-in will need to be installed).

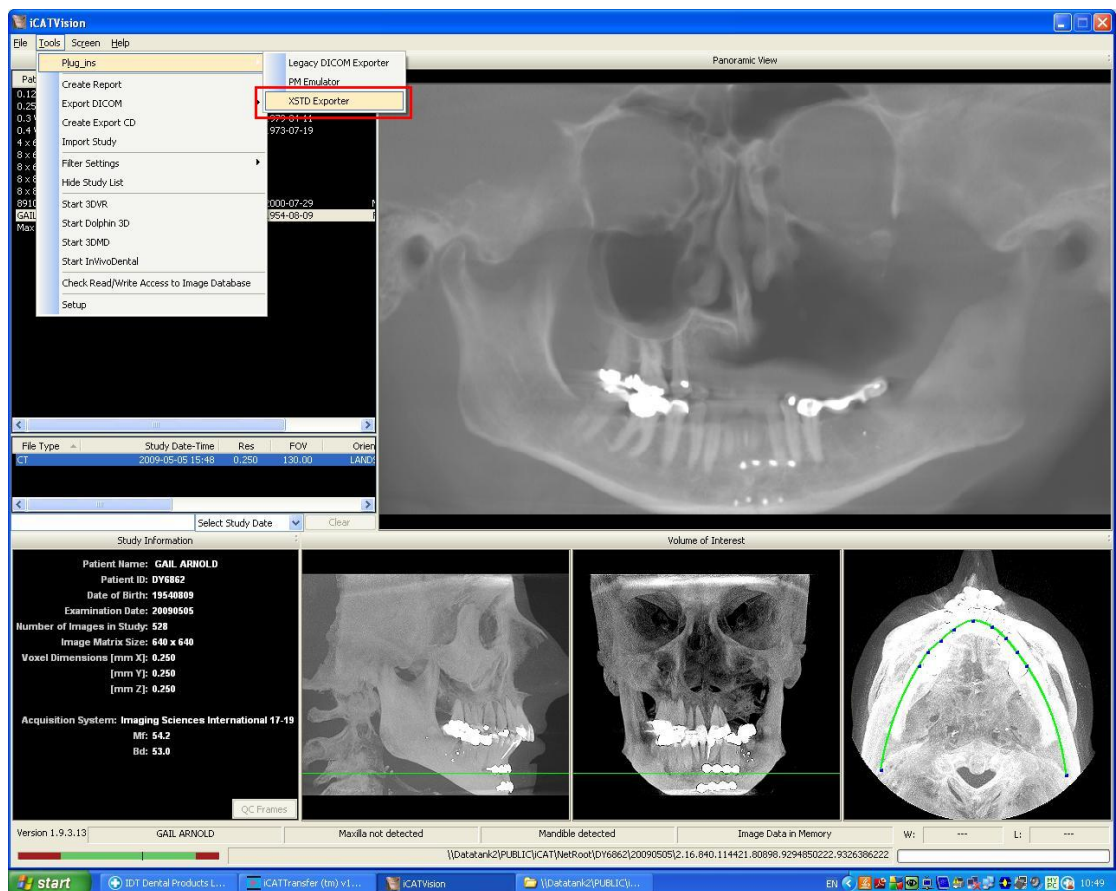

Step 3


The following dialogue box will open:

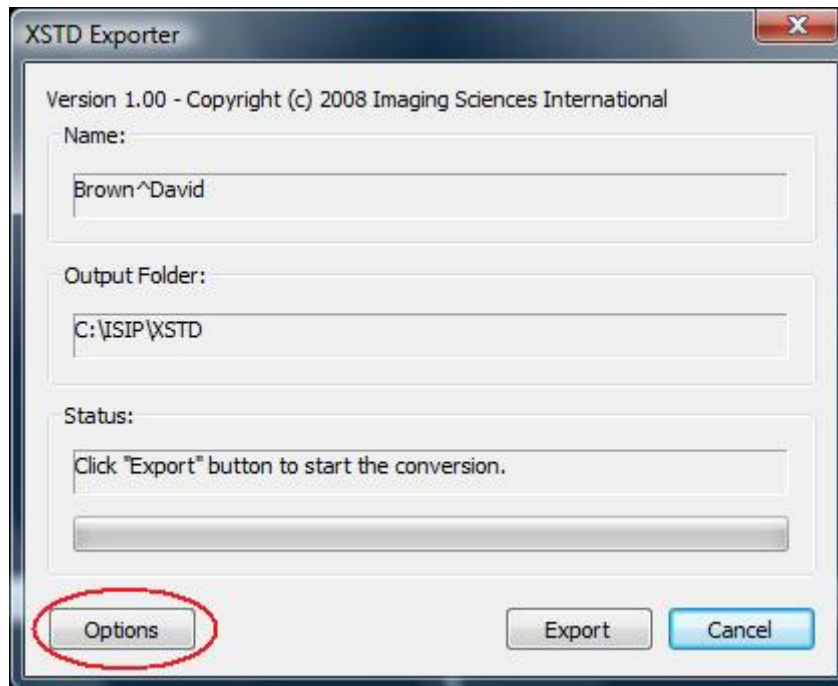

The Output Folder is C:\ISIP\XSTD in this example. To select a different Output Folder, click 'Options' and then 'Browse'.


Step 4

Note the path to the Output Folder and click **Export**:

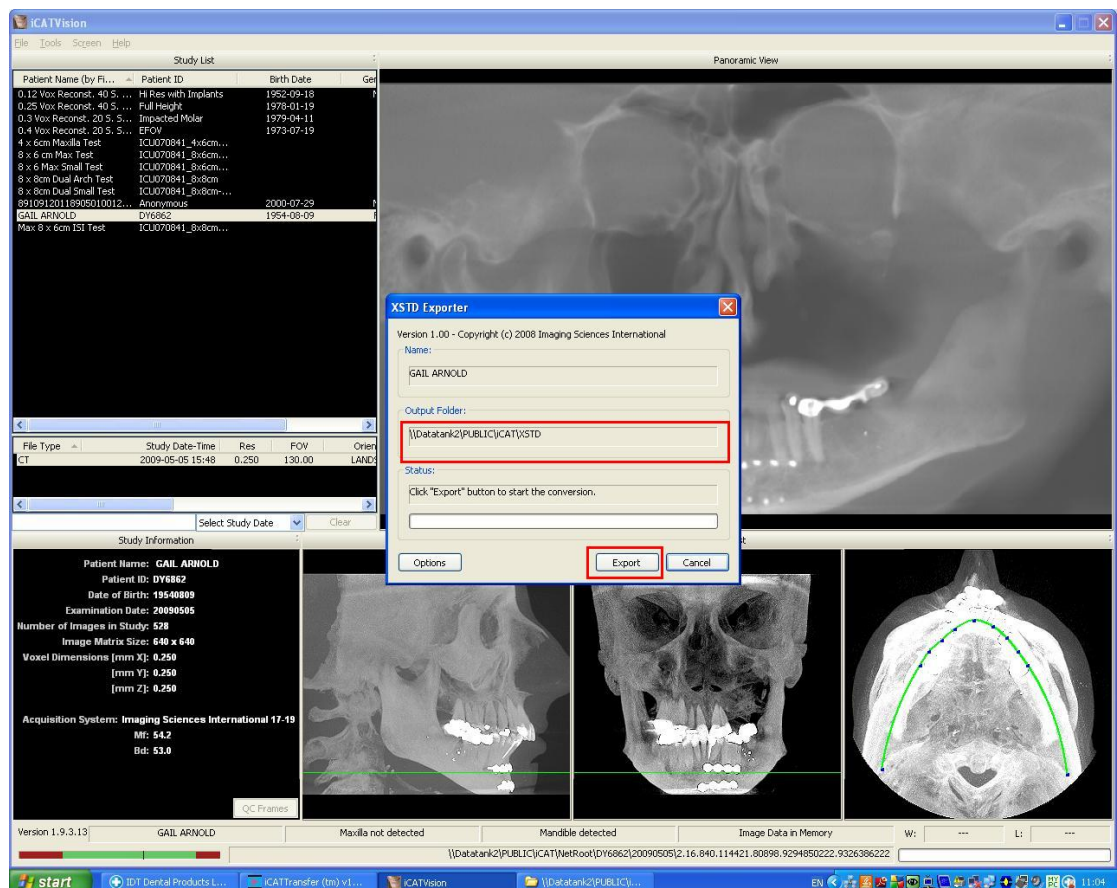

Step 5

The following dialogue box will appear while the images are being exported:

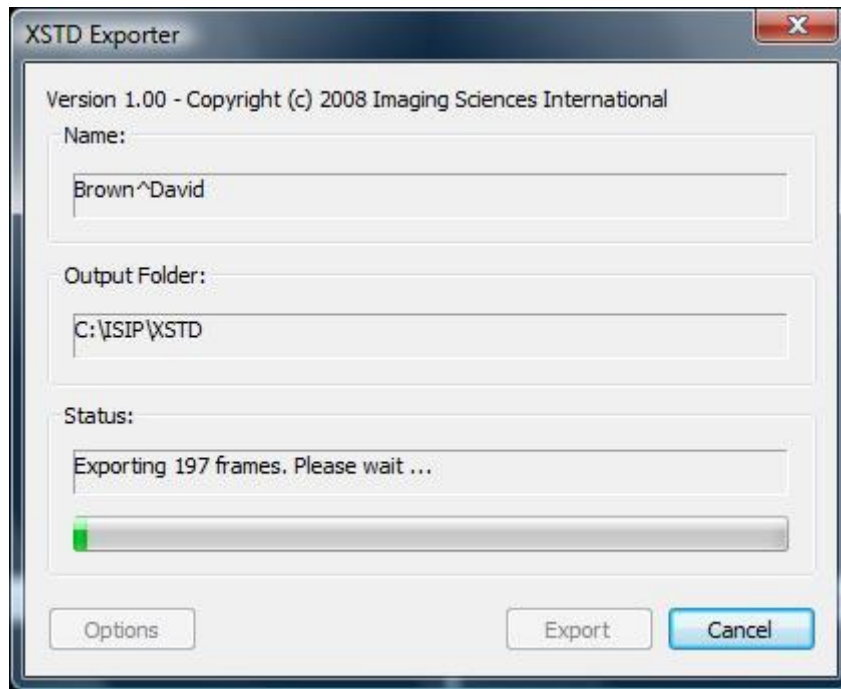

Step 6 (optional)

Close i-CATVision and navigate to the Output Folder (C:\ISIP\XSTD in this example) to find the .xstd file you just created.

Double click the .xstd file and Xoran will launch automatically and display your images.

Alternatively - launch Xoran manually - select **Import** and browse to the Output Folder to import the .xstd file.