


# How to install and use Onis Free DICOM Viewer from a Zip File you received from IDT Scans

If your images are in a Patient Folder that has already been unzipped you may skip to Step 3.

## Step 1


Save the zip file to your Desktop and double-click it. A dialogue box will open:

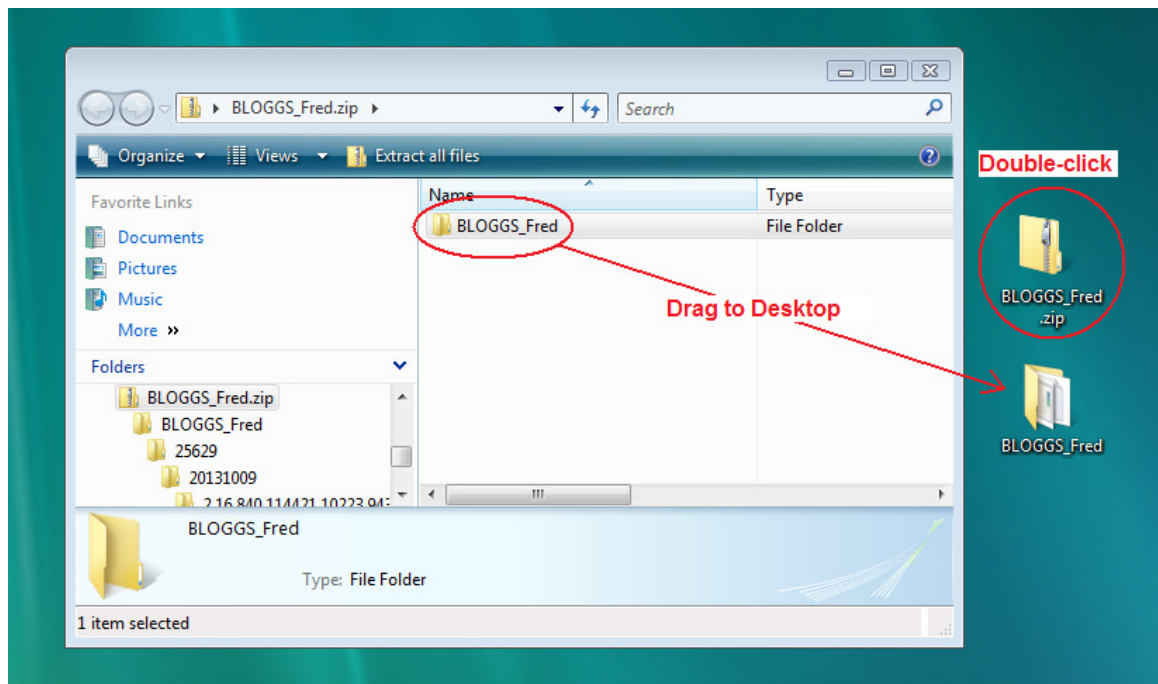

Drag the Patient Folder to your Desktop and close the dialog box. Delete the zip file from your Desktop.

## Step 2

Double-click the Patient Folder (not the zip file) to open it:

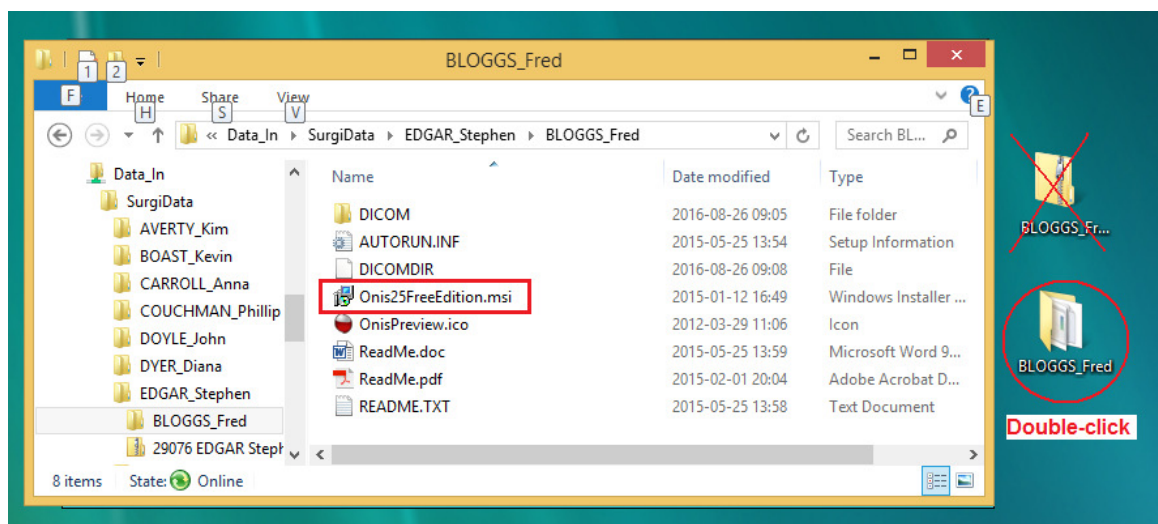

### Step 3

Double-click on “**Onis25FreeEdition.msi**”.

DICOM Viewer Onis 2.5 Installer will launch. Please click “**Run**” if you want to install Onis 2.5 Free Edition or “**Cancel**” if you already have it on your computer (in which case you may skip to Step 5).

### Step 4

Follow the onscreen instructions to install the free DICOM viewer. Click “**I accept the terms in the Licence Agreement**” if you want to install the software, or “**I do not accept the terms in the Licence Agreement**” if you do not.


### Step 5

After installation you will have an “**Onis 2.5 Free Edition**” icon on your desktop. Please double-click this icon to launch the viewer.

If a window pops up Click “**OK**” to continue or “**Visit [www.onis-viewer.com](http://www.onis-viewer.com)**” for Help.

### Step 6

The first time you launch **Onis Free DICOM Viewer** you may see this message:

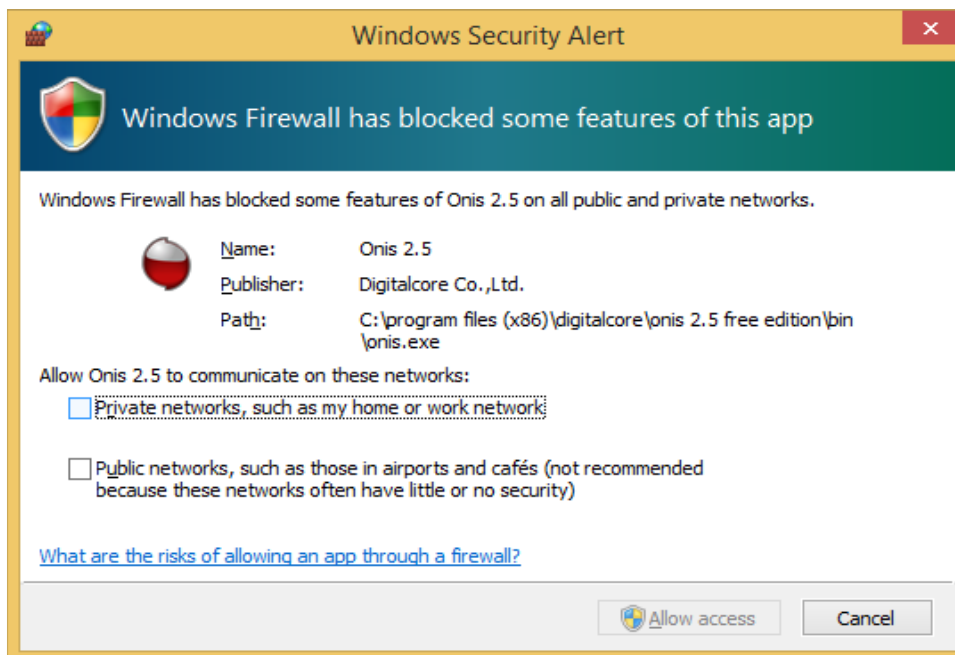


DICOM communication is only required for advanced features (not for displaying images) so we recommend clicking “**Cancel**” to block the communication.

### Step 7


If a window pops up saying “**Please insert a CD or DVD in the drive**” click “**OK**”.

Click on “**Import**” at the top of the screen, then select “**Import from a directory**”.

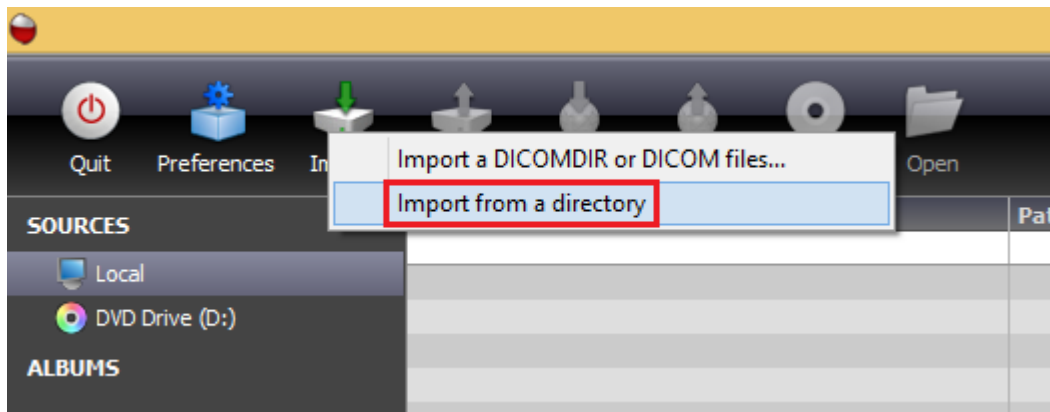

## Step 8

Navigate to the Patient Folder that you saved on your Desktop in Step 1 and select the folder with the DICOM images, then click “**OK**”.

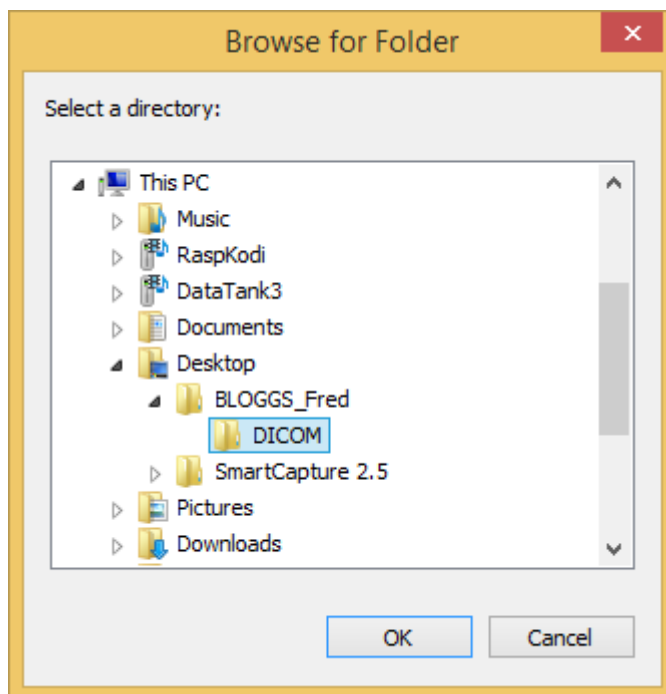

The images will be imported.

## Step 9

Patient Name will be listed and thumbnail images will be displayed. Double click the thumbnail for the images you want to view.

For further help please visit <http://www.onis-viewer.com/faq.aspx>