

How to download and install Simplant View

Simplant View is a free viewer that can be used to view CT Scans and take basic measurements. The CT images must be in Simplant format – otherwise Simplant View will not open them.

If you already have Simplant View installed on your computer you may skip to Step 6.


Step 1

Go to www.idtscans.com and click **Support** at the top of the Home Page.



Step 2

Choose 'Frequently Asked Questions'

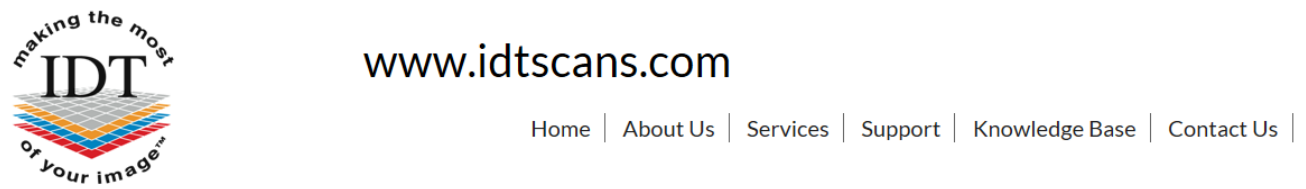

Support

Please click on the links to access the Support pages:

- [Frequently Asked Questions](#)
- [Knowledge Base](#)

Step 3

Select 'Where can I download Simplant software?' and click on 'here':



www.idtscans.com

Home | About Us | Services | Support | Knowledge Base | Contact Us |

Frequently Asked Questions

Who can service my CBCT scanner?

IDT Scans does not sell or service CBCT scanners. If you have an i-CAT or Gendex scanner please contact:

KaVo Dental Ltd.
Corinium Industrial Estate
Raans Road
Amersham
Bucks HP6 6JL
United Kingdom
Tel [+44 1494 733000](tel:+441494733000)

How can I get a SIMPLANT password?

Please click [here](#) to obtain a SIMPLANT password online from Dentsply Sirona. A CCKey is required.

Where can I download SIMPLANT software?

Please click [here](#) to access Dentsply Sirona's download page. SIMPLANT View, SIMPLANT OneShot and the Implant Libraries are free, the other versions of SIMPLANT require a licence to be paid for.

Step 4

Scroll down to the end of the page and select SIMPLANT VIEW.

Additional SIMPLANT software versions

SIMPLANT 17 Planner

[SIMPLANT Planner – English version](#)
[SIMPLANT Planner – French version](#)
[SIMPLANT Planner – German version](#)
[SIMPLANT Planner – Italian version](#)
[SIMPLANT Planner – Russian version](#)
[SIMPLANT Planner – Spanish version](#)

SIMPLANT 17 View

[SIMPLANT View – English version](#)
[SIMPLANT View – French version](#)
[SIMPLANT View – German version](#)
[SIMPLANT View – Italian version](#)
[SIMPLANT View – Russian version](#)
[SIMPLANT View – Spanish version](#)

SIMPLANT 17 OneShot

[SIMPLANT OneShot – English version](#)
[SIMPLANT OneShot – French version](#)

Step 5

Follow the on-screen instructions to install Simplant View software.


Step 6

Simplant View looks for images in the **C:\DentalData** folder.

If you want to open a Simplant file using Simplant View you need to save the Simplant file to the C:\DentalData folder first. Please see the document “Downloading and Saving Simplant files” for instructions on how to do this.


Step 7

Launch Simplant View by double-clicking the Simplant View icon on your Desktop.

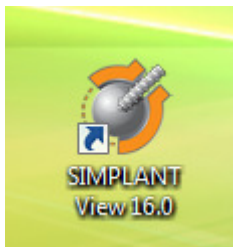

Step 8

From the **File** menu select **Open Files ...**

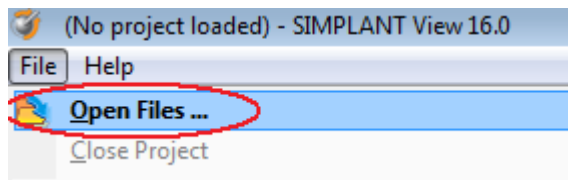

Step 9

A dialog box will open and you will see your patient listed. Double-click on your patient to view the images.