


How to download and save SIMPLANT files

SIMPLANT software looks for files in the **C:\DentalData** folder so you should download and save them in this folder. The way to do this depends on which Web Browser you are using.

Step 1


Left click on the link to access your file:

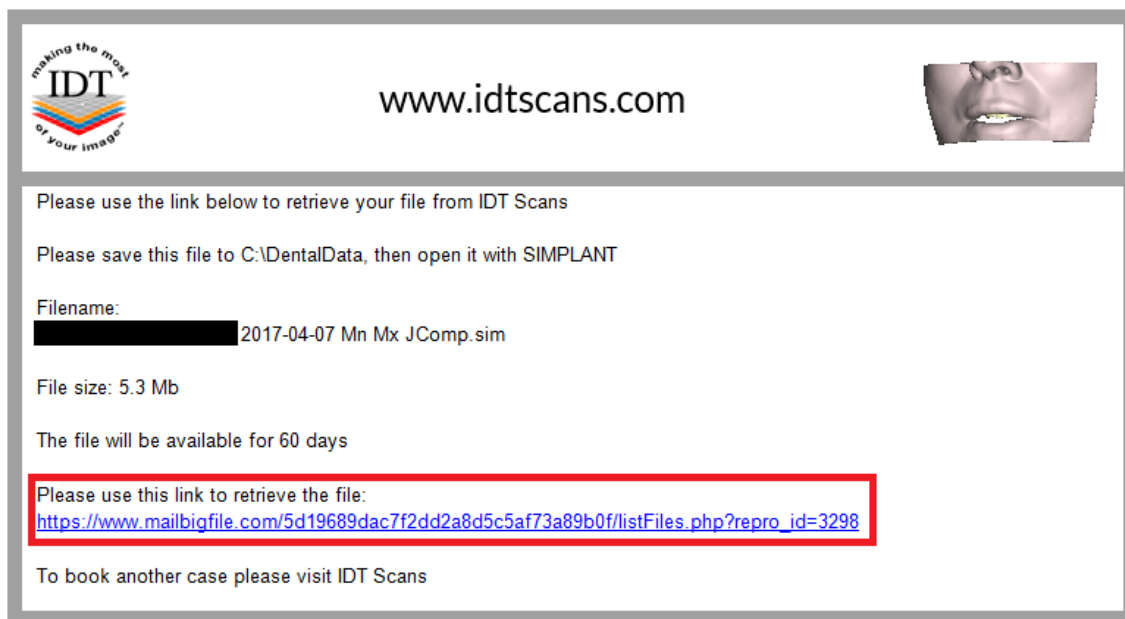

The screenshot shows the IDT Scans website interface. At the top left is the IDT logo with the tagline "making the most of your image". The URL "www.idtscans.com" is centered at the top. On the top right is a small image of a person's face. Below the header, there is a message: "Please use the link below to retrieve your file from IDT Scans". This is followed by instructions: "Please save this file to C:\DentalData, then open it with SIMPLANT". The file details are listed: "Filename: [redacted] 2017-04-07 Mn Mx JComp.sim" and "File size: 5.3 Mb". A note states "The file will be available for 60 days". A red box highlights the download link: "Please use this link to retrieve the file: https://www.mailbigfile.com/5d19689dac7f2dd2a8d5c5af73a89b0f/listFiles.php?repro_id=3298". At the bottom, it says "To book another case please visit IDT Scans".

Step 2

Right click (with the **right** mouse button) on the filename:

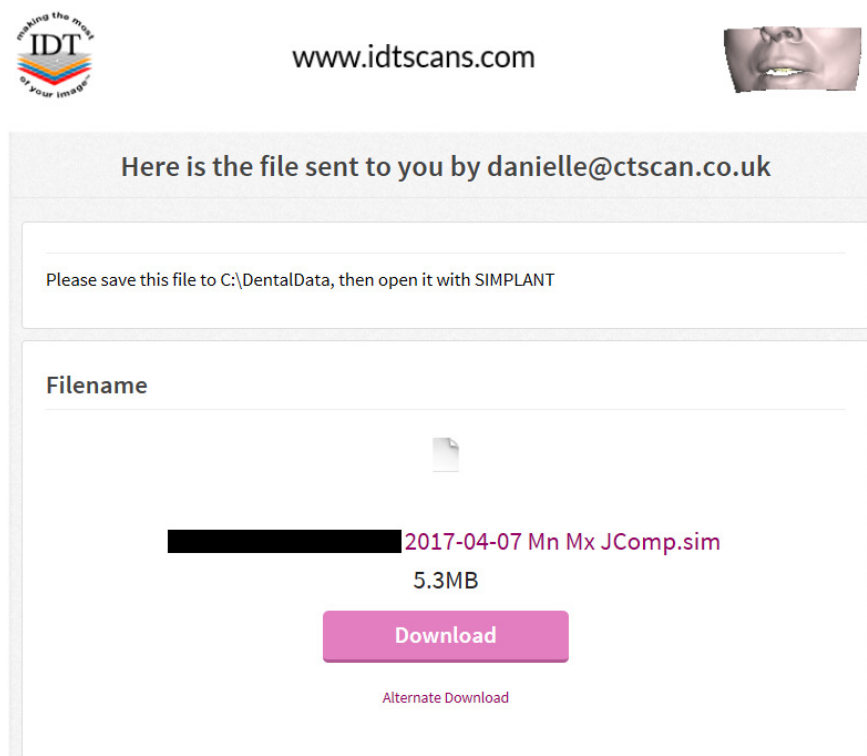


The screenshot shows the IDT Scans website interface. At the top left is the IDT logo with the tagline "making the most of your image". The URL "www.idtscans.com" is centered at the top. On the top right is a small image of a person's face. Below the header, there is a message: "Here is the file sent to you by danielle@ctscan.co.uk". Below this is a message: "Please save this file to C:\DentalData, then open it with SIMPLANT". The file details are listed: "Filename" followed by a redacted filename "2017-04-07 Mn Mx JComp.sim" and "5.3MB". There is a pink "Download" button and a smaller "Alternate Download" link below it.

If you are using Internet Explorer:

Step 3A

Left click **Save target as ...**

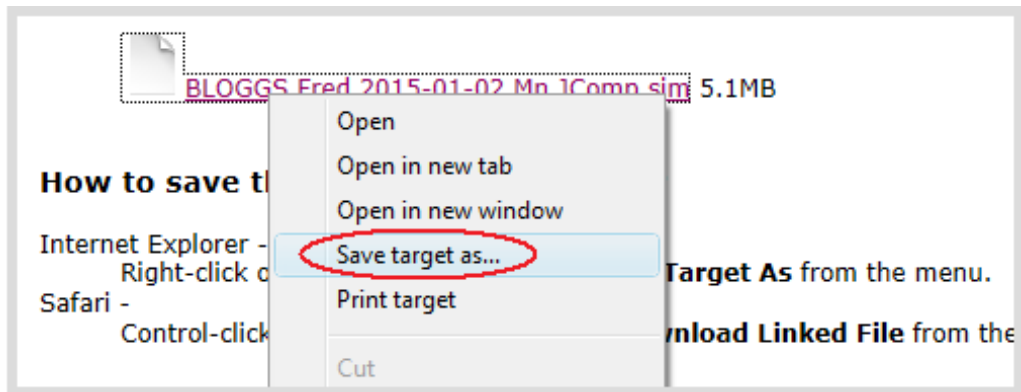


Please skip to Step 4.


If you are using Firefox:

Step 3B

Left click **Save link as ...**

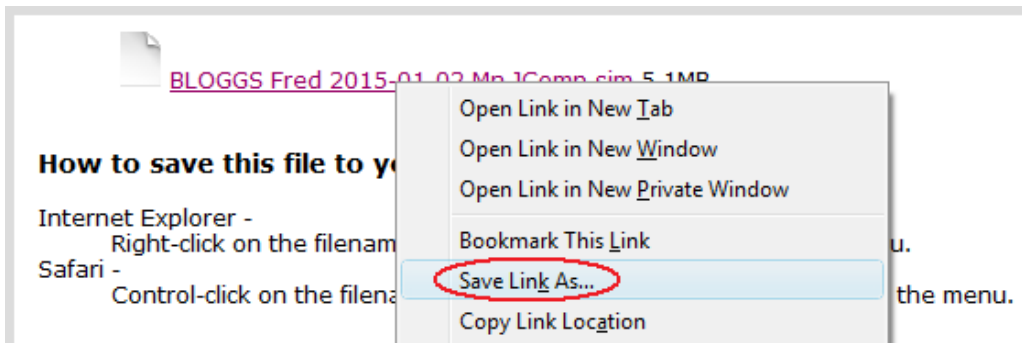

Left click the small arrow to the left of **Browse Folders**

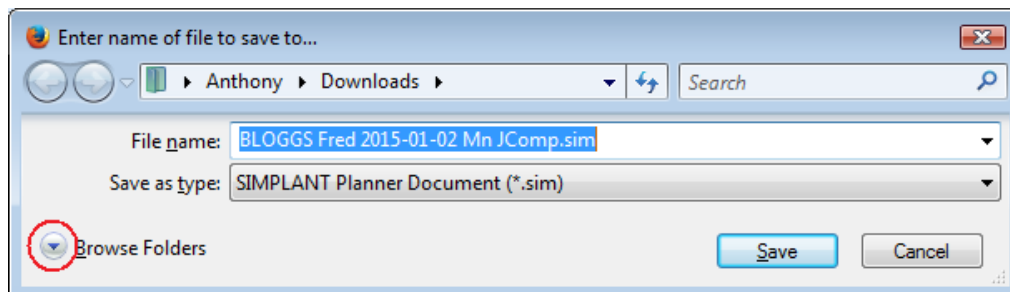


Please skip to Step 4.


If you are using Google Chrome:

Step 3C

Left click **Save link as ...**

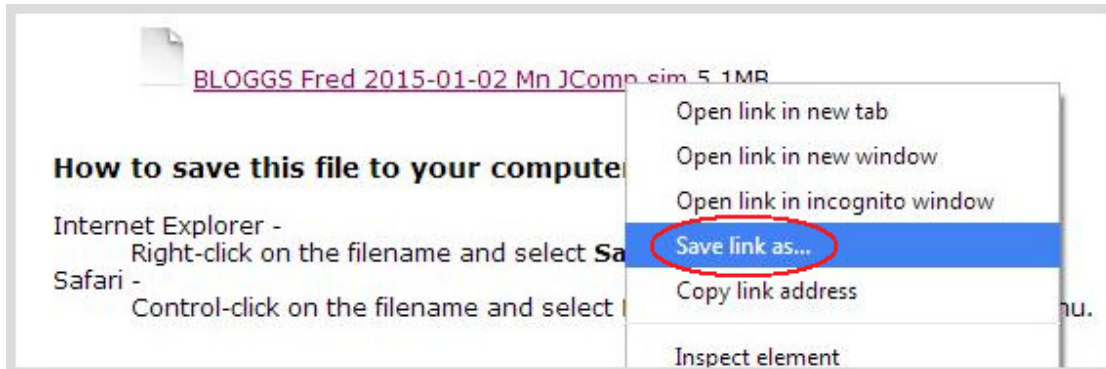


Left click the small arrow to the left of **Browse Folders**

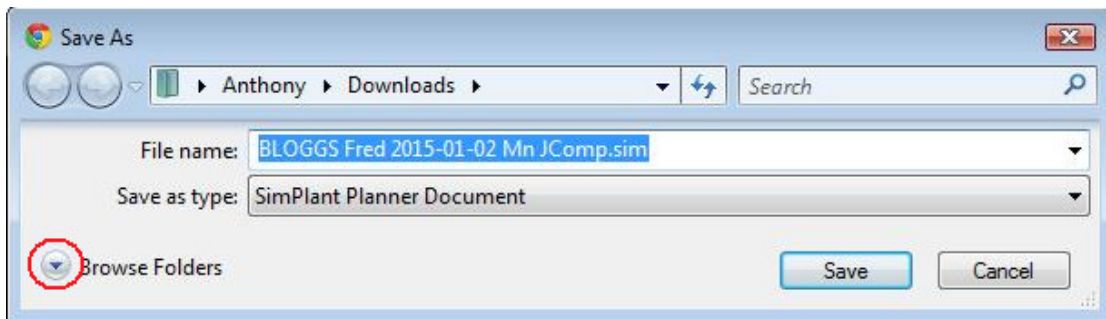

Step 4

Navigate to **Computer** then **Local Disk (C:)** then **DentalData** then click **Open**

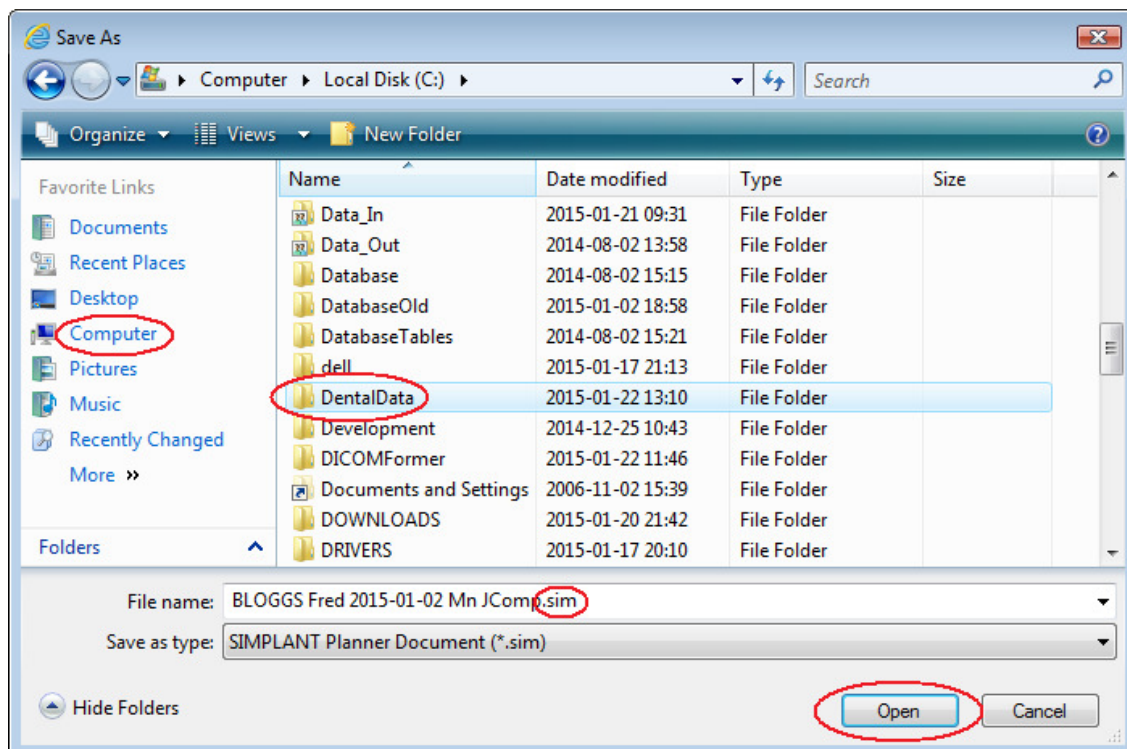

Please Note: Some websites will change the filename extension “.sim” to “.zip”.

If this happens you must rename the file as follows:


SIMPLANT Planner: change **.zip** to **.sim**

SIMPLANT Pro: change **.zip** to **.spr**

SIMPLANT OneShot: change **.zip** to **.s1s**



Step 5

Click **Save** to save your file in the C:\DentalData folder.

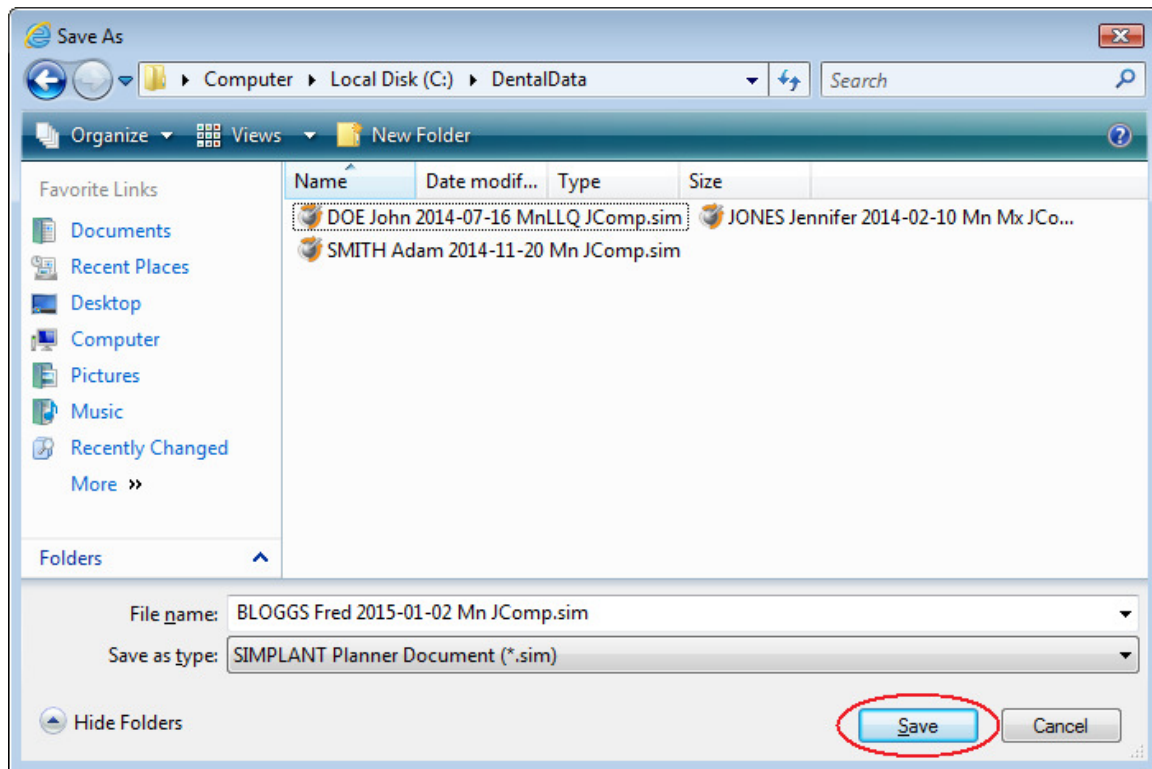

Step 6

Launch SIMPLANT by double-clicking the SIMPLANT icon on your Desktop.
From the **File** menu select **Open Files ...**

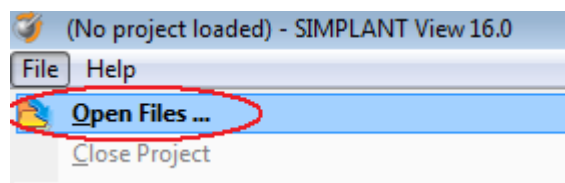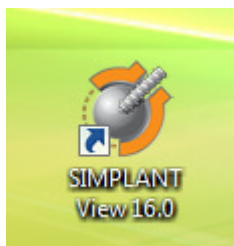

A dialog box will open and you will see your patient listed. Double-click on your patient to view the images.