

Downloading and installing Simplant View

Simplant View is free software that can be used to view CT Scans and take basic measurements. The CT images must be in Simplant format – otherwise Simplant View will not open them.

If you already have Simplant View installed on your computer you may skip to Step 6.

Step 1


Go to www.idtscans.com and click **Support** at the top of the Home Page.



The screenshot shows the IDT website homepage. At the top left is the IDT logo with the tagline "making the most of your image". To the right is the website URL "www.idtscans.com" and a navigation menu with links for "Home", "About Us", "Services", "Support" (highlighted with a red box), "Knowledge Base", and "Contact Us". Below the navigation is a large banner image of a dental professional in a white coat using a handheld device on a patient's mouth. Overlaid on the left side of the banner is a list of services: "REFORMAT AN EXISTING SCAN", "REQUEST A RADIOLOGY REPORT", "REQUEST A NEW DENTAL CT SCAN", "IMPLANT SIMULATION, MODELS, GUIDES", and "UPLOAD YOUR FILE NOW". At the bottom of the banner, a dark bar contains the text "Choose a scanning site in the UK or Ireland".

Step 2

Choose 'Software'

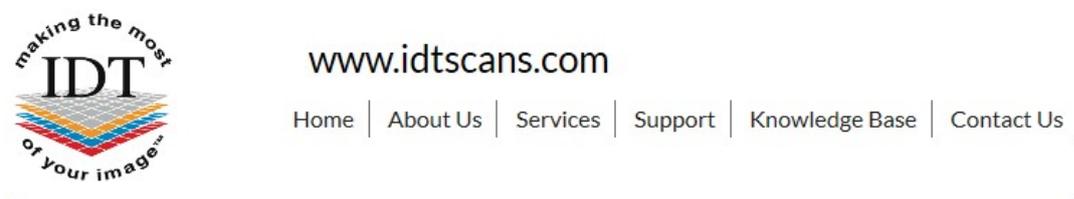

The screenshot shows the IDT website support page. At the top left is the IDT logo with the tagline "making the most of your image". To the right is the website URL "www.idtscans.com" and a navigation menu with links for "Home", "About Us", "Services", "Support", "Knowledge Base", and "Contact Us". Below the navigation is a horizontal line, followed by the word "Support" in a large, bold font.

Support

Please click on the links to access the Support pages:

- [Frequently Asked Questions](#)
- [Software](#)
- [Downloads](#)

Step 3

Choose 'Simplant View'



www.idtscans.com

Home | About Us | Services | Support | Knowledge Base | Contact Us |

Software

For information about the Simplant software family please click [here](#).

To download Simplant software please click on the following links. You may have to scroll down several pages to find the version you need. Some files are very large and may take a long time to download.

- [Simplant Pro](#)
- [Simplant View](#)
- [Simplant OneShot](#)

Step 4

Scroll down to find **Simplant Installers** then **Simplant 18.5** then **View**.

Click on <http://link.simplant.com/dentalsoftware/SimplantViewSetupEN.msi> to install the English version.

Simplant Installers

Simplant 18.5

Pro:

- <http://link.simplant.com/dentalsoftware/SimplantProSetupFR.msi>
- <http://link.simplant.com/dentalsoftware/SimplantProSetupEN.msi>
- <http://link.simplant.com/dentalsoftware/SimplantProSetupIT.msi>
- <http://link.simplant.com/dentalsoftware/SimplantProSetupES.msi>
- <http://link.simplant.com/dentalsoftware/SimplantProSetupDE.msi>

View:

- <http://link.simplant.com/dentalsoftware/SimplantViewSetupFR.msi>
- <http://link.simplant.com/dentalsoftware/SimplantViewSetupEN.msi>
- <http://link.simplant.com/dentalsoftware/SimplantViewSetupIT.msi>
- <http://link.simplant.com/dentalsoftware/SimplantViewSetupES.msi>
- <http://link.simplant.com/dentalsoftware/SimplantViewSetupDE.msi>

Step 5

Follow the on-screen instructions to install Simplant View software.

Step 6

Simplant View looks for images in the **C:\DentalData** folder.

If you want to open a Simplant file using Simplant View you need to save the Simplant file to the **C:\DentalData** folder first.

Please see the document “**Downloading and Saving Simplant files**” for instructions how to do this.


Step 7

Launch Simplant View by double-clicking the Simplant View icon on your Desktop.



Step 8

From the **File** menu select **Open Files ...**

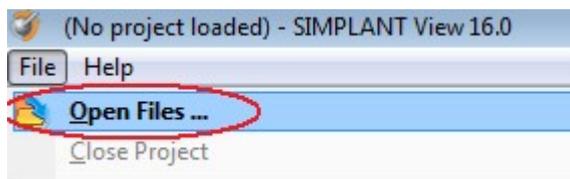

Step 9

A dialog box will open and you will see your patient listed. Double-click on your patient to view the images.